FR

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Any serious incident connected to the use of Audispray Ultra should be reported to the manufacturer and to the competent authority of the Member State where the user lives.

EN

EARWAX PLUGS**FORM**

Ear spray: Non-pressurized 20 ml bottle with a dosing pump, usable from ages 3 and up.

PROPERTIES

The dioctyl sodium sulfosuccinate breaks down the components of the wax in the earwax plug or build-up. The ethoxydiglycol dissolves the lipids in the wax.

INDICATIONS

Treatment of earwax plugs or build-ups. Facilitates the removal of wax impactions by the doctor.

METHOD OF ADMINISTRATION

To eliminate an earwax plug or build-up: 2 sprays twice a day, during 5 days. Before having a wax impaction removed by a physician: 4 sprays in the ear canal, 15 to 30 minutes before the procedure.

CONTRAINDICATIONS

Should not be used in the event of an ear infection or in general, on any person who has or has had ear problems (dizziness, medical or surgical history...).

Do not use, if allergic to any of the components.

INSTRUCTIONS FOR USE

- Point the nozzle towards the ear and spray 2 times into the ear canal, keeping the bottle in a vertical position
- Massage the base of the ear for a few seconds
- Let the product act for 15 to 30 min
- Tilt the head to the side to evacuate the excess and wipe
- Rinse the nozzle with hot water after each use

PRECAUTIONS FOR USE

Avoid contact especially with the ocular mucosa, if contact occurs rinse with water thoroughly.

If symptoms persist, consult a doctor.

AUDISPRAY ULTRA is a medical device.

Any serious incident connected to the use of Audispray Ultra should be reported to the manufacturer and to the competent authority of the Member State where the user lives.

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